



WEAR & CARE INSTRUCTIONS

Fabric Supports

Unit 2, Royds Close, Royds Lane, Leeds, LS12 6LL
T: 0113 231 1800 F: 0113 231 1805
E: info@taycare.com W: www.taycare.com

Fabric Supports

This device is intended for the exclusive use of the named patient.

This device conforms to the essential requirements set out in Annex 1 of the Medical Devices Directive 93/42/EEC. Any requirements not met are listed.

Use this leaflet in conjunction with any instructions given to you by any medical professional.

- ♦ It is not intended that you become dependant on your support.
- ♦ Wear your support for short periods of time, gradually increasing wear as you become more familiar with it.
- ♦ If your support is designed to control your abdomen it is recommended that you apply whilst lying down.
- ♦ Fasten the straps firmly from bottom to top.
- ♦ Wash your support regularly in warm soapy water after removing any stiffeners. Remember to refit the steels in the position in which they came from.
- ♦ Rinse the support thoroughly and allow to dry flat.
- ♦ The support can be ironed when slightly damp, but DO NOT apply heat to the elastic.

When your support needs repairing, wash it and take it to the Orthotic Department, identified with your name and address.

If you are concerned with any aspect of your support, please contact the Orthotic Department immediately to seek professional medical advice.

Orthotist name and contact number: