



WEAR & CARE INSTRUCTIONS

Protective Headwear

**Unit 2, Royds Close, Royds Lane, Leeds, LS12 6LL
T: 0113 231 1800 F: 0113 231 1805
E: info@taycare.com W: www.taycare.com**

Protective Headwear

This device is intended for the exclusive use of the named patient.

This device conforms to the essential requirements set out in Annex 1 of the Medical Devices Directive 93/42/EEC. Any requirements not met are listed.

Use this leaflet in conjunction with any instructions given to you by any medical professional.

- ◆ Your headwear is NOT a safety device, it is designed to provide additional protection.
- ◆ The material of construction should be handled with care, as it can be easily damaged by solvents or other chemicals.
- ◆ Wipe your headwear with a damp cloth or sponge and leave to dry naturally.
- ◆ Check your headwear regularly for signs of damage or excessive wear.

If your headwear needs repairing, take it to the Orthotic Department, identified with your name and address.

If you are concerned with any aspect of your headwear, please contact the Orthotic Department immediately to seek professional medical advice.

Orthotist name and contact number: