



WEAR & CARE INSTRUCTIONS

Orthotic Inlays

**Unit 2, Royds Close, Royds Lane, Leeds, LS12 6LL
T: 0113 231 1800 F: 0113 231 1805
E: info@taycare.com W: www.taycare.com**

Orthotic Inlays

This device is intended for the exclusive use of the named patient.

This device conforms to the essential requirements set out in Annex 1 of the Medical Devices Directive 93/42/EEC. Any requirements not met are listed.

Use this leaflet in conjunction with any instructions given to you by any medical professional.

- ◆ Wear your Inlays for short periods of time, gradually increasing wear as you become more familiar with it.
- ◆ Wear your inlays in combination with suitable footwear, as instructed by your medical professional.
- ◆ Your inlays will eventually wear out, which you will notice by the re-occurrence of foot pain. When this happens contact the Orthotic Department to make an appointment.
- ◆ Wipe the material when dirty with a damp cloth or sponge.
- ◆ Periodically check your Inlays for any signs or wear.

If your Inlays need repairing, take them to the Orthotic Department, identified with your name and address.

If you are concerned with any aspect of your inlays, please contact the Orthotic Department immediately to seek professional medical advice.

Orthotist name and contact number: